 **PLAYMAKER** | WITH QUIETVIBE™ TECHNOLOGY

# MVP*mini*

**Percussion Massage System**



Instruction Manual  
**Model # MVP-500**



# TABLE OF CONTENTS

Description.....	4
Warnings .....	5
Contents .....	6
Features .....	7
Application .....	8
Charging the Battery .....	9
Operating Instructions.....	10
Maintenance & Storage .....	10
Technical Specifications .....	11
Product Warranty.....	12
Registering Your Device .....	13
Warranty Registration Card .....	14

## **PlayMakar® MVPmini Percussion Massage System**

The MVPmini Percussion Massage System featuring QUIETVYBE™ Technology and our patent-pending ProTips™ is a lightweight, ultra-compact device designed to loosen knots, reduce fatigue, and melt away stress. It comes with four ProTips™ interchangeable massage heads to enhance your treatment.

### **What is Percussion Massage?**

Percussion massage delivers a barrage of short rapid strokes penetrating deep into the muscle tissue to release lactic acid, pain, and tension. It helps improve blood circulation, increase range of motion, stimulate muscle growth, enhances recovery, and decrease stress.

### **What is Vibration Massage?**

Vibration massage uses a stroke that presses and releases in an up and down movement on a group of muscles. It can be effective in soothing or stimulating soft tissues depending on the applied speed and level of pressure.

## **IMPORTANT SAFETY INSTRUCTIONS**


- The MVPmini is intended for adult use only and should be used as instructed in this manual.
- Do not use the MVPmini on children or infants.
- This device is for use on soft tissues areas. It should not be used on the head, spine, or other bony parts.
- The MVPmini should be used on clean, dry body parts only.
- If you experience pain, discomfort, or bruising, discontinue use immediately.
- Do not use the MVPmini on contusions, rashes, irritated skin, bruised, or injured areas of skin.
- Keep the device away from your hair to avoid entanglement.
- Avoid using the MVPmini under a pillow or blanket to avoid excessive heating.
- Never immerse the device in water.
- Keep ventilation opening free of dust and debris.
- Do not attempt to disassemble, dismantle, alter, or remove the battery from the MVPmini. Doing so could result in damage to the device and will void the warranty.
- The MVPmini should only be charged using the USB-C cable supplied with this device.

## CONTENTS

- PlayMakar® MVPmini Percussion Massager
- USB-C Charging Cable
- ProTips™ with EchoFlex™ Technology:
  - Shoc™ Absorption & Reverberation Massage Head
  - EchoBall™ 1.5" Absorption & Reverberation Massage Head
  - ImpactBall™ 2" Massage Head
  - AcuTrigger™ Acupressure Massage Head



## FEATURES

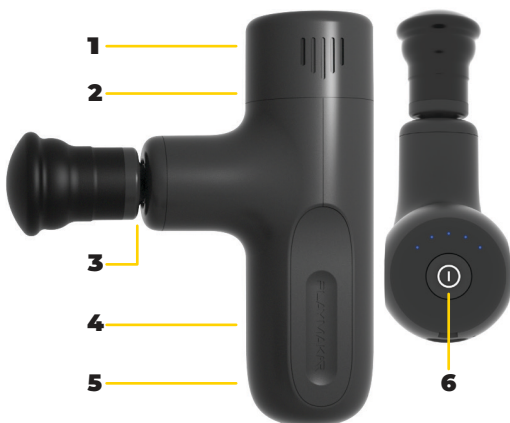

1	Powerful brushless motor
2	Low noise treatment with QUIETVYBE™ Technology >45db
3	7 mm Stroke length
4	Ergonomic grip built for comfort
5	2600mAh Lithium battery for up to 5 hrs. of therapy per charge
6	5 Adjustable Speeds (1800, 2100, 2400, 2700, & 3000 RPMs)

## APPLICATION



### **Shoc™ Absorption & Reverberation Massage Head**

The Shoc™ Head replicates the deep pressure of a thumb while the patent-pending air transfer pocket absorbs impact and reverberates for a more comfortable experience. It is ideal for sore, tender, or bony areas, and overall use.



### **EchoBall™ Absorption & Reverberation Massage Head**

The 1.5" EchoBall™ Head provides a medium to high impact treatment to sore, achy muscles. It is perfect for use on dense muscle groups.



### **ImpactBall™ 2" Massage Head**

The 2" ImpactBall™ Head is designed for general use on large and small muscle groups. It covers a larger surface area and delivers a lower impact than the EchoBall™ Head for a broader treatment option.



### **AcuTrigger™ Acupressure Massage Head**

The AcuTrigger™ Head simulates acupressure therapy for painful trigger points. It is also ideal for use on smaller muscle areas including the hands, elbows, and feet.



## CHARGING THE BATTERY

1. Plug the cable into the opening located near the bottom of the MVPmini.
2. Plug the USB-C connector into a USB port to begin charging.
3. A green light by the charging port will illuminate to indicate the device is charging.
4. Charging is complete when the green light shuts off.

**Important:** Please allow ~3 hours for the MVPmini to fully charge before use.

## BATTERY CAPACITY INDICATORS

> 30% Blue light

< 30% Red light

< 5% Red light will blink 5 times

## OPERATING INSTRUCTIONS

1. Attach the desired massage head securely to the device.
2. Press the ( I ) button on the bottom of the device for 2-3 seconds to start your massage treatment on the lowest intensity level.
3. Press and release the ( I ) button to increase the intensity to the next level. Repeat until the desired intensity is reached.

**Important:** The number of blue lights indicates the level of intensity from 1 - 5.

4. The device will shut off automatically after 10 minutes of use. Restart the device to continue your treatment.
5. Press the ( I ) button through all 5 intensity levels to turn off the device.

## MAINTENANCE & STORAGE

- Use a dampened cloth to wipe the device after each use.
- Do not use harsh cleaning on the device or accessories.
- The device should be stored in a dry, ventilated environment.
- Charge the battery once a month when storing the device for long periods.

## TECHNICAL SPECIFICATIONS

**Vibration Frequency:** 1800-3000 RPM

**Stroke Displacement:** 7 mm

**ProTips™ with EchoFlex™:** 4

**Battery Indicator Mode:** LED Display

### RPM BY INTENSITY LEVEL:

Level	RPM (Revolutions Per Minute)
1	1800
2	2100
3	2400
4	2700
5	3000

## BATTERY SPECIFICATIONS

**Battery Type:** 18650 Power Type 3C

**Rated Voltage:** 11.1V

**Charging Time:** ~2-3 hrs.

**Cycle Life:** 800 cycle (>80%;/ (Capacity > 80%)

## CHARGER SPECIFICATIONS

**Model Number:** MX24W1-2401000U

**Input Voltage:** 100/240V AC

**Output Voltage:** 12V DC

**Rated Power:** 25 W

**Charging Current:** ~1000mA

## **WARRANTY**

This PlayMakar® MVPmini Massager comes with a 1-year warranty from the date of purchase. If the product fails due to a defect in material or workmanship, PlayMakar will repair or replace the product or necessary components.

An extended warranty is provided for devices registered at [PlayMakar.com/register](http://PlayMakar.com/register)

Consumable items such as massage heads and AC adapters are guaranteed to be free from defects at the time of delivery.

The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alterations, or disassembly by unauthorized individuals.

PlayMakar® reserves the right to replace or repair the unit at our discretion.

### **PlayMakar, Inc.**

Attn: Warranty  
5825 Park Vista Circle  
Fort Worth, Texas 76244

### **[www.PlayMakar.com](http://www.PlayMakar.com)**

Phone: 833-PLAY-MAKAR or 833-752-9625

Email: [IAM@PlayMakar.com](mailto:IAM@PlayMakar.com)

## THANK YOU FOR PURCHASING

We appreciate you choosing the PlayMakar® MVPmini Massager. We believe our product is the best device on the market and think you will, too. Our team is passionate about sports, fitness, and helping athletes of every level meet their performance goals with our exceptional products.

Your satisfaction is important to us. If you have questions please contact our Customer Service Representatives. They can be reached by phone at 833-752-9625 between 8 a.m. to 4 p.m. CST, Monday through Friday, or by email at [IAM@PlayMakar.com](mailto:IAM@PlayMakar.com).

## REGISTERING YOUR DEVICE

Please register your device within 14 days of purchase to receive product offers, discounts, and instructional videos. Registration may be completed online or by mailing the attached Warranty Registration Card.

Go to [PlayMakar.com/register](http://PlayMakar.com/register) to register your product online. Use the following information to complete your registration:

**Model Number:** MVP-500

**Serial Number:** Located on the bottom of your device

## Warranty Registration Card

Product Name		Model Number	
Date of Purchase	Place of Purchase		
Serial Number			
Name			
Address 1			
Address 2			
City	State	Zip	
Country			
Phone			
Email			
How did you hear about PlayMakar?			
Why did you choose your PlayMakar product?			

Please return this completed card in a stamped envelope within 14 days of purchase to receive additional offers and product information.

Mail to: PlayMakar® Products  
5825 Park Vista Circle  
Fort Worth, Texas 76244







5825 Park Vista Circle  
Fort Worth, TX 76244

(833) 752-9625  
[IAM@PlayMakar.com](mailto:IAM@PlayMakar.com)  
[www.PlayMakar.com](http://www.PlayMakar.com)